

How To Scan A Case With A Bite

In your scanning system, ensure that True Function is selected as your lab partner.

1. Take a physical bite registration as needed for the appliance you are prescribing. Suggested techniques include George Gauge, Phonetic, Airway Metrics, etc. There is no need to send this physical bite once you have scanned it, however it is great to keep as a reference in case there is a discrepancy when the device is produced.
2. Trim bite registration with scalpel or similar instrument following the occlusal plane to expose as much buccal tooth structure as possible (try to expose cusp tips and incisal edges).
3. Proceed to scan arches as required by your scanning system.
4. Place trimmed bite registration in patient's mouth and take a bite scan (ensure arches align on the monitor as they align with the physical bite in the patient's mouth).
5. Submit the case.

NEW!!! There is no need to send a separate RX now.

Submit your RX and instructions 2 ways:

1. Through your scanning software once you have a signature on file with us.
2. Through our FREE online ordering system (DDX). Contact us today to get set up!

Unsure about the bite registration, technique, appliance, etc.... don't hesitate to call!
(619) 466-1872 or info@truefunction.com

Providing peace of mind by producing quality products that are true to fit and function.